

# Masimo SafetyNet™

Radius PPG™

## Setup Instructions

**Warning:** Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

### GET TO KNOW YOUR SYSTEM



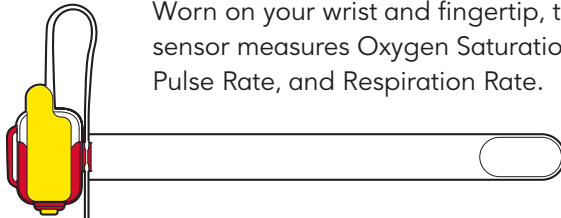
#### Masimo SafetyNet App

Masimo SafetyNet displays Oxygen Saturation, Pulse Rate, Respiration Rate and other data from Radius PPG. It also sends data to your care provider.

#### Wearable Sensors

##### Radius PPG™ Sensor

Worn on your wrist and fingertip, the sensor measures Oxygen Saturation, Pulse Rate, and Respiration Rate.



##### Radius PPG™ Chip

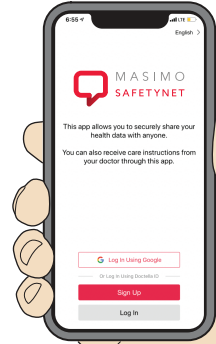
Snapped into the wrist portion of the sensor, the chip sends your data to the Masimo SafetyNet App.



## Getting Started

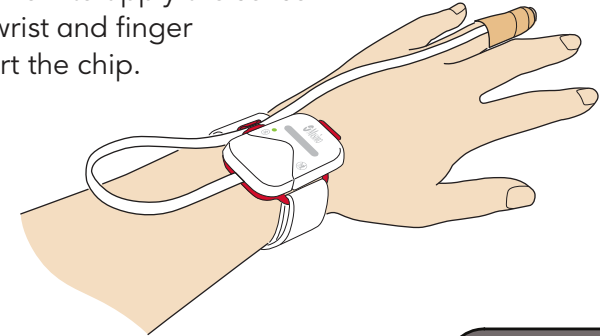
### 1 Set-up Masimo SafetyNet App

Follow instructions from the email or text message invitation to download the Masimo SafetyNet App and create an account.



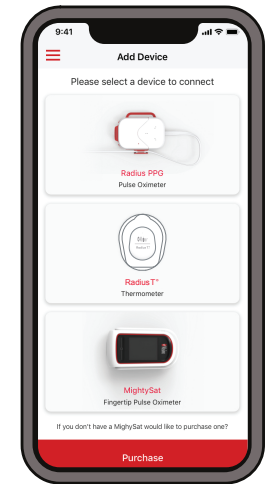
### 2 Apply Sensor

See instructions on the back showing how to apply the sensor to your wrist and finger and insert the chip.



### 3 Pair Sensor

Under Masimo SafetyNet App **Settings**, enter the **Add Device** menu, and follow instructions for pairing the Radius PPG sensor to your phone.

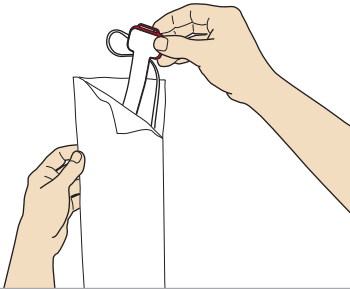
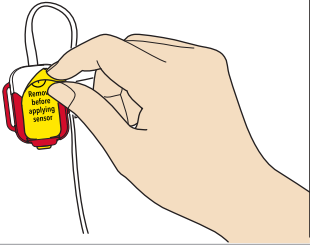
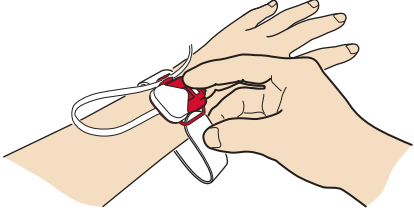
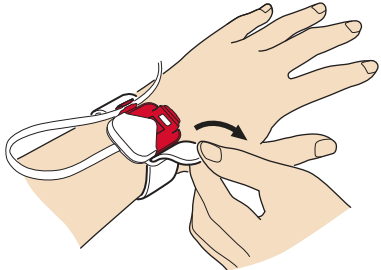


For additional resources, including videos and instructions, visit [mymasimo.com](http://mymasimo.com)

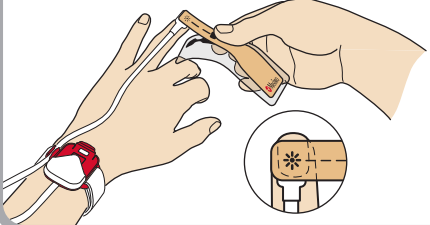


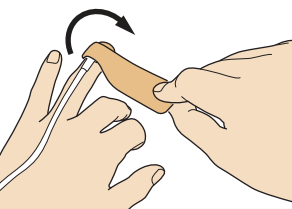
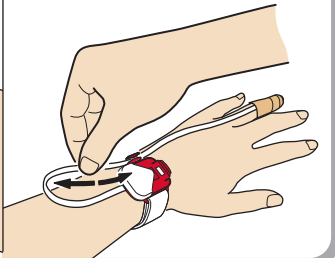
see back for sensor application →

# How to Apply your Sensor

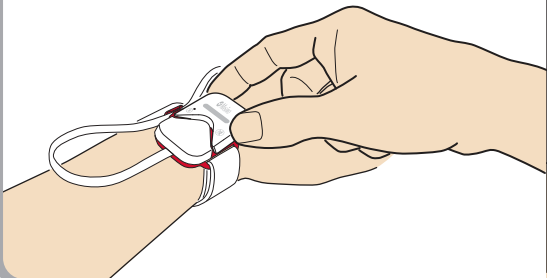
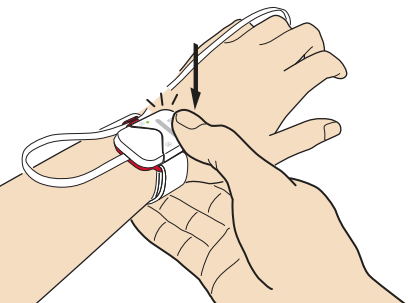
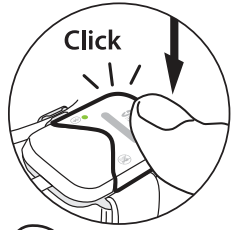
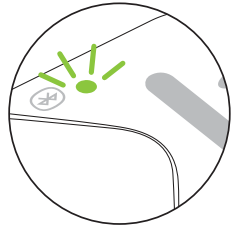
## 2.1 Sensor Set Up

<p>1. Remove sensor from pouch</p> 	<p>2. Peel off yellow label to activate battery</p> 	<p>3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop</p> 	<p>4. Wrap the loose end of the strap around your wrist and press</p> 
--	---	--	---

## 2.2 Apply Finger Sensor

<p>1. Remove part of the plastic film and place star (✱) on top of fingernail of ring finger</p> 	<p>2. Wrap halfway around finger. Make sure the star and square are lined up directly across from each other</p> <p>Place black square (■) on finger pad</p> 	<p>3. Squeeze top and bottom of wrap to secure</p> 	<p>4. Remove plastic film and continue to wrap remainder around finger</p> 	<p>5. Adjust cable to comfortable length</p> 
---	--	---	---	---

## 2.3 Insert Chip

<p>1. Insert the chip into the open compartment on the sensor</p> 	<p>2. Press down on the chip to secure</p> 	<p>Click</p>  <p>Listen for click to confirm chip placement</p>	 <p>Look for light to confirm chip connection to sensor</p>
---	---	--	--